 agrolabo S.p.A.

DERMAKIT™

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*Dermatophyte detection kit based on D.S.M.
(Dermatophytes Selective Medium)*

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INTRODUCTION The most common cause of dermatophytosis are fungi belonging to the genera *Microsporum*, *Trichophyton* and *Epidermophyton*. Cases of dermatophytosis are clinically similar to many other skin diseases. Therefore the diagnosis must be confirmed before beginning antifungal treatment.

THE KIT The culture medium used in DERMAKIT allows the selective and specific identification of dermatophytes within 72 hours, directly on samples taken in the surgery or in the field, allowing a specific diagnosis to be obtained in a few days.

DERMAKIT is an high quality diagnostic test, protected against contaminations and enriched by the following components:

- Specific nutrients that facilitate the growth of dermatophytes
- Selective antibiotics which prevent the growth of non-pathogenic saprophytic mycetes and bacteria
- Colour pH indicator that change color (from orange to red) in presence of dermatophytes. The action of the specific pH indicator used, is based on the characteristic of *Microsporum*, *Trichophyton* and *Epidermophyton* genera produce alkaline metabolites, that induce the indicator colour change.

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COMPARATIVE STUDY BETWEEN DERMAKIT SPECIFICITY AND MICROSCOPIC EXAMINATION IN DOG, CAT AND HORSE SAMPLES, USING ONLY COLOUR CHANGE TO DETERMINE THE DERMAKIT RESULT.

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
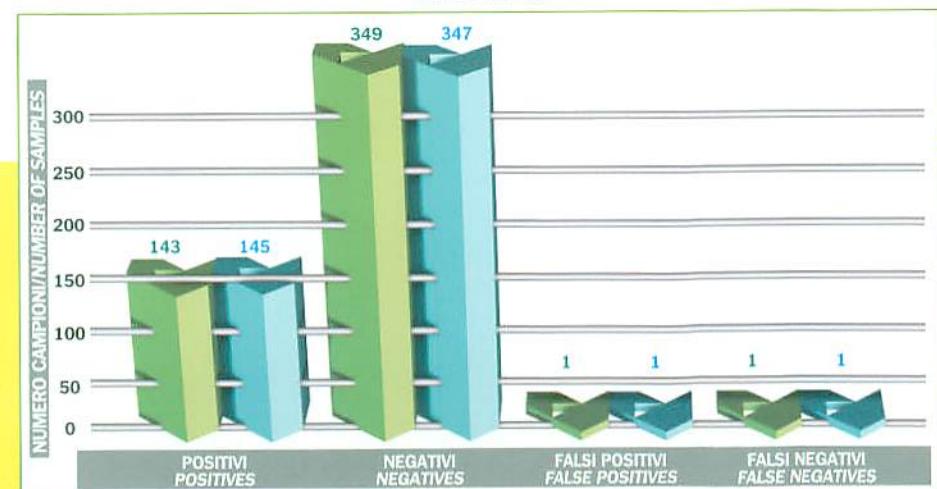
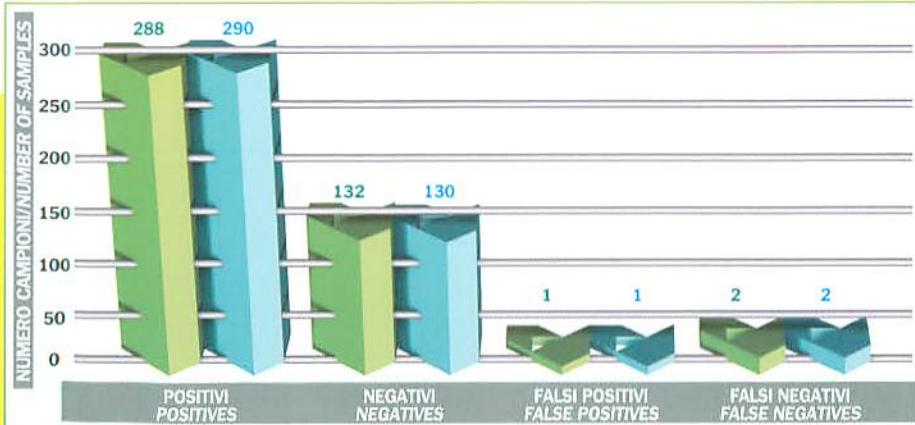
DIFFUSIONE DEI DERMATOFITI IN CANI, GATTI, CAVALLI e CONIGLI.
**FREQUENCY OF PRESENCE OF VARIOUS DERMATOPHYTES
 IN DOGS, CATS, HORSES and RABBIT.**

SPECIE/SPECIES	CANI/DOGS	GATTI/CATS	CAVALLI/HORSES	CONIGLI/RABBITS
<i>M. canis</i>	++++	++++	++	++
<i>M. gypseum</i>	+++	++	+++	+++
<i>M. audouinii</i>	+			+
<i>M. distortum</i>	++	+	++	
<i>T. mentagrophytes</i>	++	++	++	++++
<i>T. verrocosum</i>	+		++	
<i>T. equinum</i>	+		++++	
<i>T. rubrum</i>	+			
<i>T. gallinae</i>	+			
<i>T. schoenleinii</i>	++	++	+	+

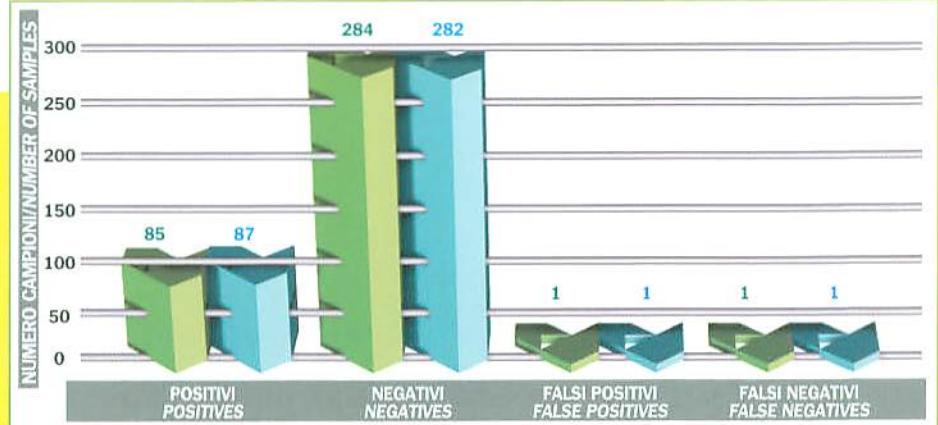GATTO

Order and Information, only for Italy,
call free of charge

www.agrolabo.it

E-Mail: for info: agrolabo@agrolabo.it • for orders: acquisti@agrolabo.it

HEAD OFFICE, RESEARCH AND PRODUCTION LABORATORIES: Via Masero snc - 10010 Scarmagno (TO) Italy - Tel.: +39 (0)125-731111 - Fax: +39 (0)125 731190